



# ReMedium Aqua

Sodium Hyaluronate / Sterile Pre-filled  
2ml x 2 syringes






## What is the Aquashine procedure?

The Aquashine procedure is one of the ways to rejuvenate the skin with the help of a uniform and direct injection of hyaluronic acid into the dermis layer, which leads to the brightness and moistening of the skin, concentrating the moisture, since the HA has a moistening effect.

In total, the 10-15 minute procedure results in skin rejuvenation, which becomes brighter, moisturized and more elastic, as it is filled with a lot of moisture.

## What is HA (hyaluronic acid)?

Hyaluronic acid is one of the natural polysaccharides, an extracellular matrix ingredient, a natural substance existing in the human body. It replenishes the vigorous source of cellular tissue, which leads to the protection and non-aging of the skin. HA contains 200-300 times more moisture than the human body, since it is one of the safest ingredients of the human body, it helps moisturize and rejuvenate the skin.

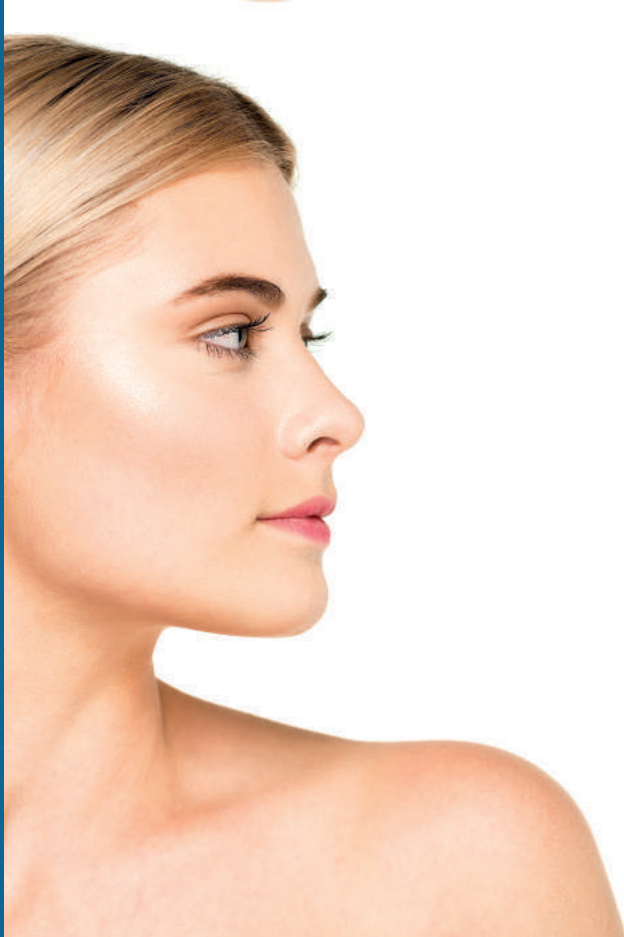

## The Effect of the Aquashine procedure

- Skin rejuvenation**
  - brighten the skin
  - textured skin
  - the elasticity of the skin
  - pulling up the pins
  - removal of pigments (freckles, spots)
  - addition to makeup
- Against aging**
  - removal of fine wrinkles
  - improve the elasticity of the skin
  - prevents the aging of the skin
- Humidification**
  - improve the condition of the skin
  - improve the skin condition regardless of the change seasons
  - remove dead skin cells
- Effect**
  - appeared effect after 1-2 weeks
  - during 3-4 months

## ReMedium Aqua INJECTION


Unstabilized hyaluronic acid get purified to be suitable for mesotherapy after steriled through gel chromatography method of "Shiseido". ReMedium Aqua injection can help plump up the skin, protect them from the harmful effects of free radicals, improve blood microcirculation, and normalize the complexion.

The drug is injected at a depth of 1 mm into the surface layers of the epidermis.

1. Treatment Area
2. Treatment Procedure
3. Injection Interval
  - 1) 1st stage : Inject first 2 treatments by every 3-4 weeks
  - 2) 2nd stage : Inject every 3 months after 1st stage
    - You can expect continuous effects.



## Areas of use

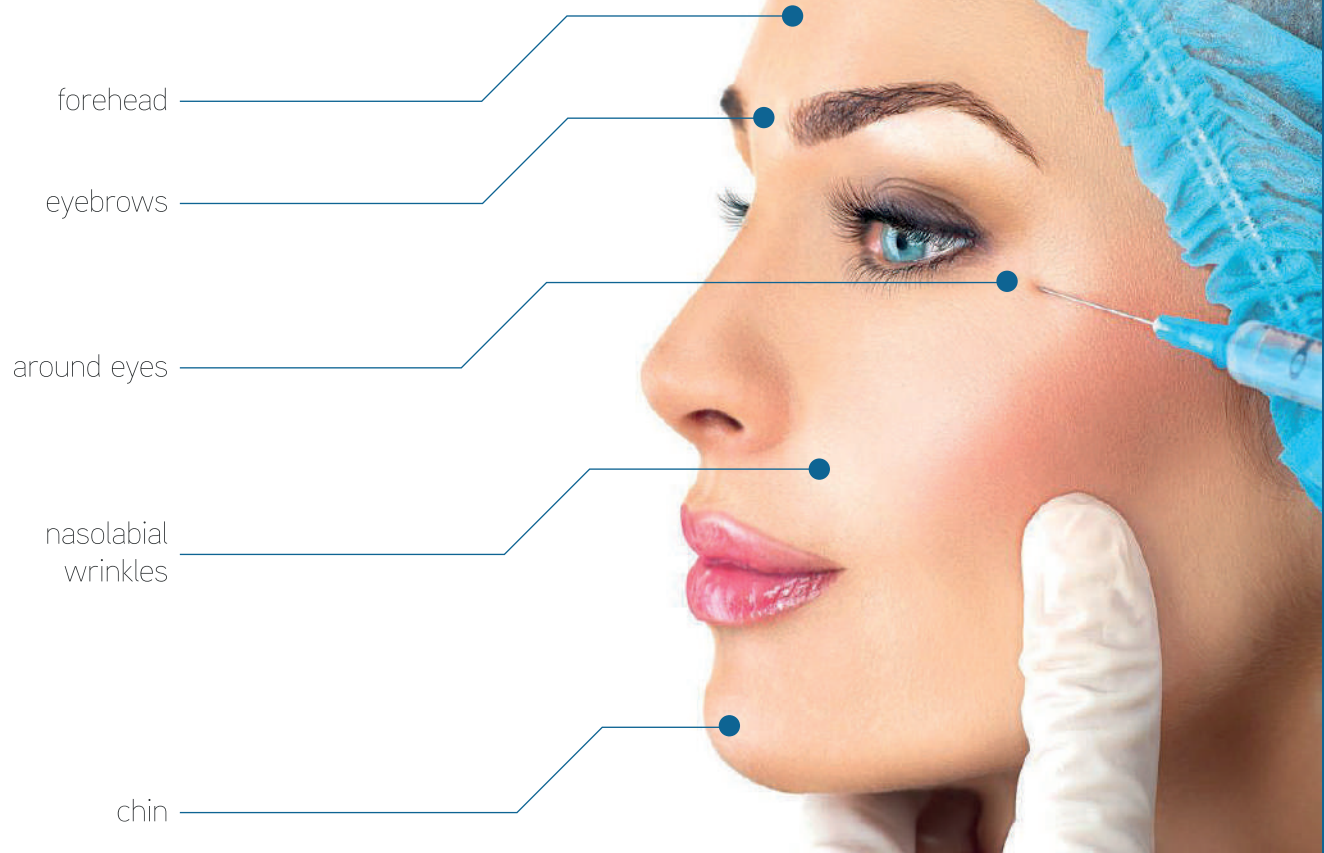



## Method of procedure



Anesthetize the skin




Insert ReMedium Aqua injection with a needle  
or an injection device evenly



If necessary,  
apply other care products

## Results of the procedure



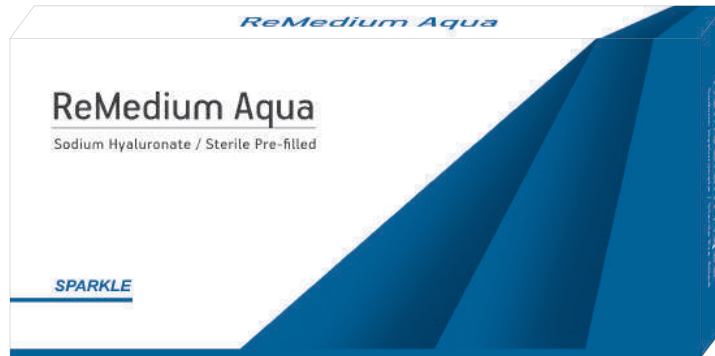

## LINE UP

- Component  
Non cross-linked Hyaluronic acid 20mg/mL
- Purpose of Use  
Temporal correction of Facial Wrinkles
- Model Name & Packaging Info  
2 pre-filled syringe / 1box  
ReMedium Aqua / 2.0mL
- Storage  
Room temperature, No light, Do not Freeze
- Shelf Life  
3 Years



## Treatment Caution


1. There may be temporal redness, bruise, swell in the injected area after treatment.
2. Avoid touching the injected area with contaminated hands.
3. Face wash gently after 3-4 hours and make-up can be worn after 24 hours.
4. Avoid massage, sauna, extreme work-out and sunlight.
5. Keep in room temperature and avoid light.




# Certificates



Memo

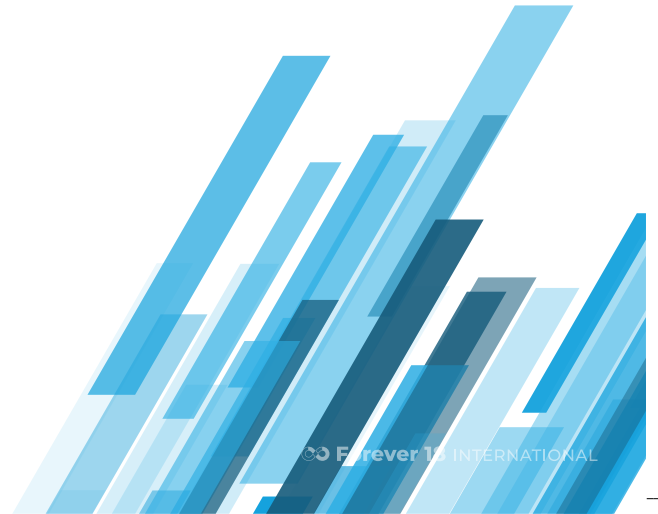

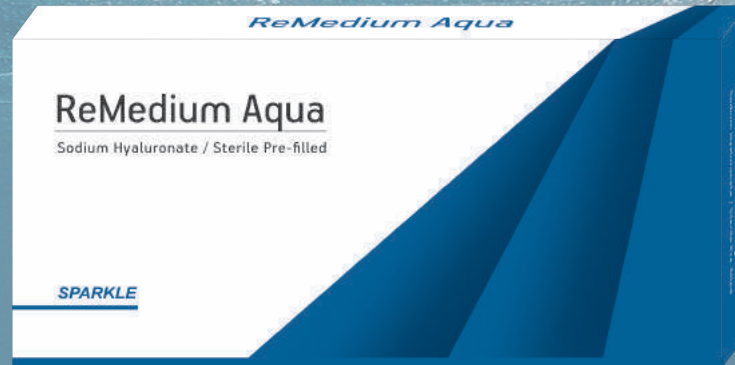

## ReMedium Aqua

Sodium Hyaluronate / Sterile Pre-filled

 **Forever 18** INTERNATIONAL  
#1001, 384, Sagajeong-ro, Jungnang-gu, Seoul, Korea

Tel. 82-70-7525-1021 Fax. 82-2-2088-1029

[www.remediumkorea.com](http://www.remediumkorea.com)  
[contact@remediumkorea.com](mailto:contact@remediumkorea.com)